Quick Start Guide: AMG & EMG

AMG
Acceleromyography

EMG
Electromyography

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
Sponsors

Australia


Teleflex Medical Australia
Level 4, 197 Coward St
Mascot NSW 2020

Caution


Federal (US) law restricts this
device to sale by or on the order
of a physician.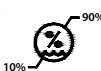

www.xavant.com/university

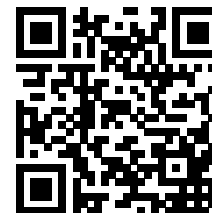www.xavant.com/docs

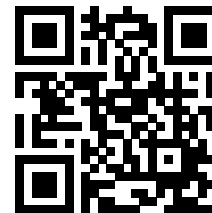English	4
Czech (Čech)	7
Danish (Dansk)	10
Dutch (Nederlands)	13
French (Français)	16
German (Deutsche)	19
Greek (ελληνικά)	22
Italian (Italiano)	25
Japanese (日本語)	28
Korean (한국어)	31
Norwegian (Norsk)	34
Polish (Polskie)	37
Portuguese (O Português)	40
Romanian (Română)	43
Spanish (El Español)	46



Quick Start

English



NMT Monitoring: AMG Setup


1 Check Batteries & Switch Device On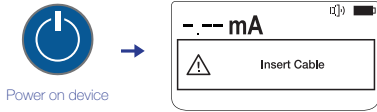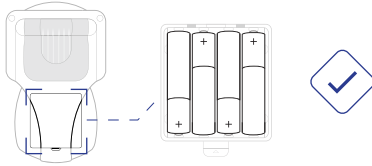


2 Connect NMT Monitoring Cable AMG

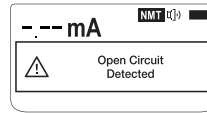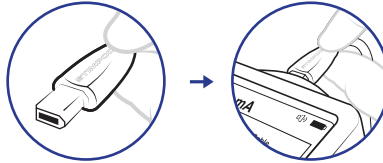
3 Prepare Patient's Skin for Monitoring

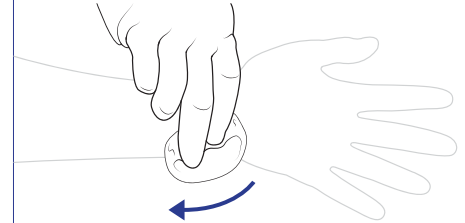4 Attach Electrodes


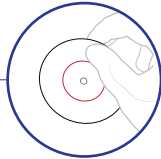
5 Connect Clips to the Electrodes

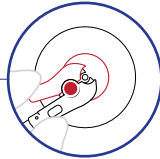6 Attach Accelerometer to Appropriate Patient Appendage

Patient Setup

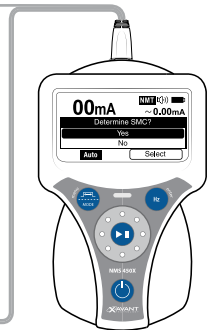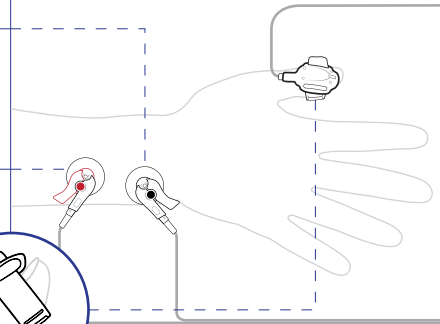
Quick Start

English



NMT Monitoring: EMG Setup


1 Check Batteries & Switch Device On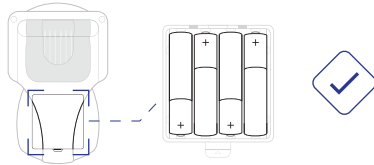


2 Connect NMT Monitoring Cable EMG

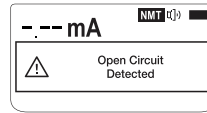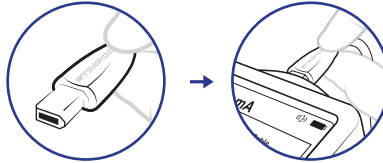3 Prepare Patient's Skin for Monitoring

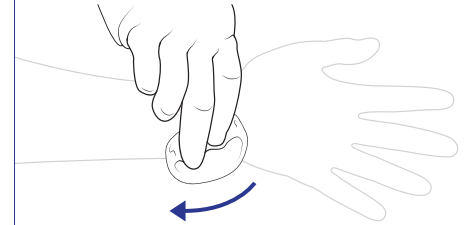4 Attach EMG Electrode to Patient

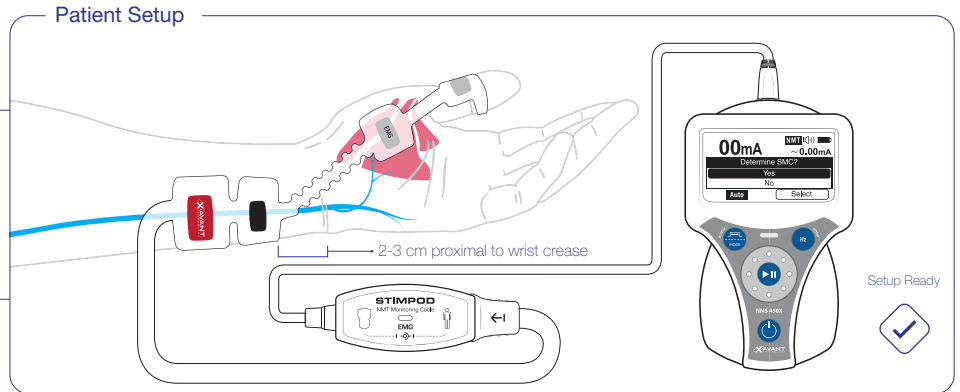5 Connect EMG Electrode to the NMT Monitoring Cable EMG

You Choose: **Patient Stimulation Options I**

Option A:

OneTouch NMT (AUTO Mode)

1 Select Auto Stimulation Mode

Confirm **SMC** and press Enter

The device screen displays '00mA' and 'NMT (M)' with a battery icon. Below the current, it says 'Determine SMC?' with 'Yes' and 'No' options. At the bottom, there are 'Auto' and 'Select' buttons. The 'MODE' button on the device is highlighted.

2 Supramaximal Current (SMC)

Determine optimal current and verify electrode placement

The device screen displays '35mA' and 'REF NMT (M)' with a battery icon. Below the current, there is a bar graph with five bars of increasing height. At the bottom, there is an 'Auto' button. The 'MODE' button on the device is highlighted.

3 Full Case Monitoring Initiated

Depth of Block automatically determined and monitored

The device screen displays '60mA' and 'REF NMT (M)' with a battery icon. Below the current, it shows '59.9mA' and '100%' with a timer '00:36'. At the bottom, there are 'Auto' and 'Recovered' buttons. The 'MODE' button on the device is highlighted.

Option B:

Manual Stimulation

1 Select Stimulation Mode

Press **MODE** to scroll through to **TOF Mode**

The device screen displays '00mA' and 'NMT (M)' with a battery icon. Below the current, there is a 'TOF' mode indicator. At the bottom, there is a 'TOF' button. The 'MODE' button on the device is highlighted.

2 Adjust current as required

Roll **Current wheel**

Press **Enter**

The diagram shows a circular 'Current wheel' with a play/pause symbol in the center. Arrows indicate it can be rolled. Next to it is the 'Enter' button, which is highlighted.

The device screen displays '60mA' and 'NMT (M)' with a battery icon. Below the current, there is a 'TOF' mode indicator. At the bottom, there is a 'TOF' button.

3 Press Play to Initiate Stimulation

Press **Play**

A large circular button with a play/pause symbol is shown.

Or

Hold in **Play button** for 2 sec for repeat stimulations

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